ru

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Luchtstroomsensor Wegwerp

REF 0517-24

BRAEBON Medical Corporation
1-100 Schneider Road
Kanata, ON, Canada, K2K 1Y2
01.613.831.6690

EC REP CEpartner4U BV
Esdoorniaan 13
3951 DB Mearn
The Netherlands
www.cepartner4u.eu

© 2022 BRAEBON® Medical Corporation

en

Intended Use

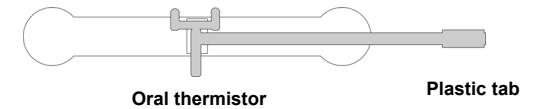
BRAEBON Disposable Airflow sensor is intended for use during sleep disorders studies as a qualitative measure of respiratory airflow for recording onto a data acquisition system.

Use the BRAEBON 0510BS Disposable cTherm and Breath Sensor Interface Cable to connect the Disposable Airflow sensor to standard polygraph equipment.

Warning: Sensors marked as single use (disposable) must NOT be cleaned and reused. Airflow sensors are for diagnostic purposes only and are not intended for use as an apnea monitor.

Caution: US Federal law restricts this device to sale by or on the order of a physician. Airflow sensors are sensitive instruments and must be handled with care. Never immerse the battery enclosure (of the 0510BS) at any time.

Nasal thermistor elements


Oral thermistor

Plastic tab

Setup

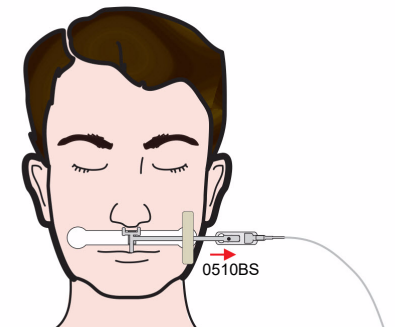1. Use an alcohol prep pad to remove any oils or lotions from the patient's skin.
2. Remove the Disposable Airflow sensor from its package.
3. Gently remove the sensor from its plastic backing.
4. Position the sensor with the two nasal thermistor prongs aligned with the patient's nostrils.
5. Press the tape against the skin to secure. Ensure the thermistor prongs are not touching the patient's skin or lips.
6. Insert the plastic tab into the 0510BS Disposable cTherm and Breath Sensor Interface Cable. Fit the metal snap through the hole on the Disposable Airflow Sensor and snap closed.
7. Place a piece of tape over the Disposable Airflow sensor to secure.

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