
BRAEBON

BRAEBON Medical
1-100 Schneider Road
Kanata, ON, Canada, K2K 1Y2
01.613.831.6690
Toll-free (N.A.): 1.888.462.4841

CEpartner4U BV
Esdoornlaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.eu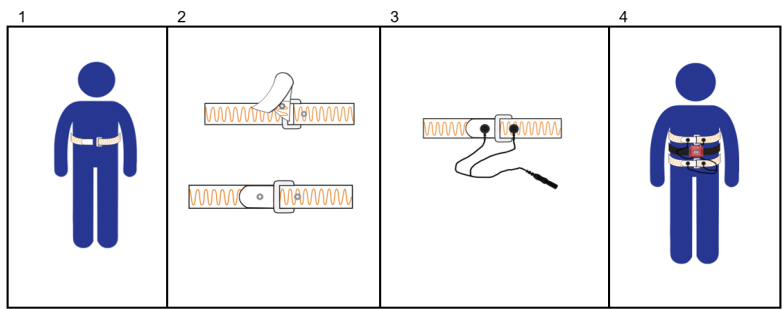

REF 5574 (45cm), 5575 (70 cm), 5576 (100 cm)

en Disposable RIP Belt

Intended Use

The Disposable RIP belt is intended to be used with a Cable for Disposable RIP Belt - (0578 (210 cm/ 7') or 8578 (37 cm/ 1.2') -during sleep disorder studies as a measure of respiratory airflow for recording onto any data acquisition system.

Warning: The Disposable RIP Belt is a single patient use (disposable) belt and not to be cleaned or reused. The Disposable RIP Belt is used for diagnostic purpose only.

Sizing: Select the BRAEBON Disposable Belt length required to fit the patient; the band can expand to 150% of its unstretched size.

Setup - using MediByte on a standard RIP belt

Wrap the belt around the patient's chest or abdomen over clothing, with the smooth side facing the patient (copper wire facing outward). Insert the Velcro-covered end through the belt loop and adjust the belt comfortably around the patient (1). Remove the Velcro backing and press the Velcro hook end to the belt (2). Connect the cable (REF 8578) to the Disposable RIP belt (3) and to the MediByte (4). Ensure the belt is neither excessively tight nor loose.

Setup - using Disposable RIP Belt with other PSG Systems: Put the belt on the patient, as described above. Connect the cable (REF 0578) to the Disposable RIP belt and to the PSG system.

EU Notice: any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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