BRAEBON

BRAEBON Medical
1-100 Schneider Road
Kanata, ON, Canada, K2K 1Y2
01.613.831.6690
Toll-free (N.A.): 1.888.462.4841

EC **REF**

CEpartner4U BV
Esdoornlaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.eu

en **RIP Belt for MediByte – All Models**

Intended Use

The RIP Belt for MediByte is intended for use during sleep disorder studies as a measure of respiratory effort. The sensor uses respiratory inductance plethysmography technology and plugs into the MediByte.

Warning: Respiratory effort sensors are for diagnostic purposes only and are not intended as an apnea (apnoea) monitor.

Caution: US Federal law restricts this device to sale by or on the order of a physician.

Note: Refer to www.braebon.com for more information. Product specifications are subject to change without notice.

How to Use

1) Belt Placement: The chest belt clips into the MediByte. Adjust the RIP Belt for MediByte to fit the patient comfortably. The chest belt should be just below or above the nipple line and the abdominal belt should be below the rib cage. To use the buckle connector, simply fasten one end of the belt into the other end of the belt. Ensure that the belt is neither excessively tight nor loose.

2) Attaching the wires: Take one of the Y-shaped black cables and plug one safety pin connector into the left socket located on the belt and the other safety pin connector into the right socket. Use medical tape to tape down the wires, if desired. Plug the keyhole connector into the black keyhole input on the bottom right of the MediByte - labeled CHEST or ABDOMEN.

Specifications

Operating Temperature: 10°C (50° F) - 35° C (95° F)

Storage Temperature: 0°C (32° F) - 50° C (122° F);

Operating/Storage Humidity: 30-95% / 10–100% Non-condensing.

Disinfection and cleaning

The RIP Belt for MediByte is classified as a non-critical item because it only contacts clothing and intact skin. Use isopropyl alcohol wipes to clean the belts and wires. Steam autoclaving is not recommended. For additional information, refer to the APIC guidelines for selection and use of disinfectants: www.apic.org.

Warranty

BRAEBON MEDICAL CORPORATION warrants to the first consumer that the RIP Belt for MediByte (the "Sensor"), when shipped in its original container, will be free from defective workmanship, performance and materials and agrees that it will, at its option, either repair the defect or replace the defective Sensor or part thereof at no charge to the purchaser for parts or labor for a time period of one year from the date of purchase. The warranty described herein shall be the sole and exclusive warranty granted by BRAEBON MEDICAL CORPORATION and shall be the sole and exclusive remedy available to the purchaser. Use of the Sensor constitutes total and complete acceptance of this warranty. Correction of defects, in the manner and for the time period described herein, shall constitute complete fulfillment of all liabilities and responsibilities of BRAEBON MEDICAL CORPORATION to the purchaser with respect to the Sensor and shall constitute full satisfaction of all claims, whether based on contract, negligence, strict liability or otherwise. In no event shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any loss of revenues or damage, direct, incidental, or consequential, including property damage, loss of profit, or personal injury resulting from the use or misuse of, or the inability to use this product. Nor shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any damages or defects in the Sensor which were caused by abuse, misuse, tampering, neglect, incorrect battery type, or repairs or attempted repairs performed by anyone other than an authorized servicer.

EU Notice: any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/ or patient is established.

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